

extriCARE® 2400 NPWT Foot Dressing Application Guide

EC2400 - F Components:

- 1 x Spoon-Shaped Dressing Pad
- 1 x Skin Drape (35x28cm)
- 1 x Suction Bell with Skirt with connection tubing (200cm) & Clamp
- 1 x Silicone Gel Strip Pack
- 4 x Nonwoven Dressing Toe Dividers




Application Instructions:

1. Clean the wound bed (according to facility protocol) with alcohol-free prep and then assess wound size and type. Apply skin sealant to the area surrounding the wound bed.

Optional: You may treat the edges of the wound with hydrocolloids for a better seal.



2. Place silicone strips in a contiguous pattern around the foot proximal to the body from the wound bed. Make sure to remove the red backing and the strips overlap each other to completely encircle the foot.

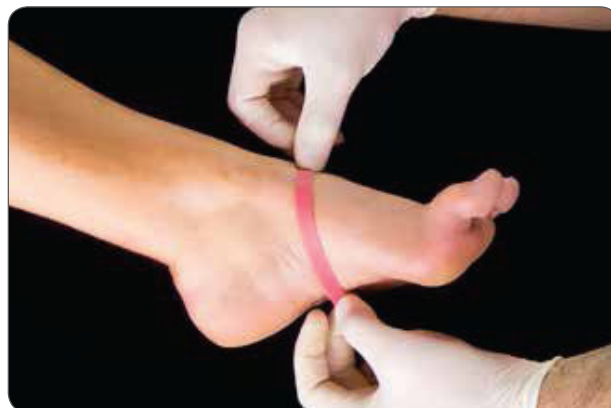

IFU30.0009 Rev. B 20140313



ALLEVA MEDICAL CO.

Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan St. Shek Mun, Shatin, N.T., Hong Kong
Tel: +852 3166 7200 Fax: +852 2355 7663 www.allevamedical.com

United States Contact:
Toll-free line at 1-877-312-NPWT
4709 Distribution Ct #2, Orlando, FL 32822
info@allevamedical.com



© 2018

Application Instructions: (Continued)

- 3.** Place the small rectangular dressing pieces between the toes. Use petroleum jelly or ointment on top of the toes to protect the toenails.



- 4.** Trim the circular portion of the dressing to fit the wound.



- 5.** Place the circular portion on the wound bed. Wrap the rectangular piece around the foot to the dorsal side. Trim the excess.



ALLEVA MEDICAL CO.


Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan St. Shek Mun, Shatin, N.T., Hong Kong
Tel: +852 3166 7200 Fax: +852 2355 7663 www.allevamedical.com

United States Contact:
Toll-free line at 1-877-312-NPWT
4709 Distribution Ct #2, Orlando, FL 32822
info@allevamedical.com



Application Instructions: (Continued)

- 6.** Apply the film over the foot by peeling back one side of the first layer. Have the patient step onto the drape so that the toes face the fold and the adhesive reaches the gel strips. Remove the remaining side of the first layer and fold over on top of the foot.

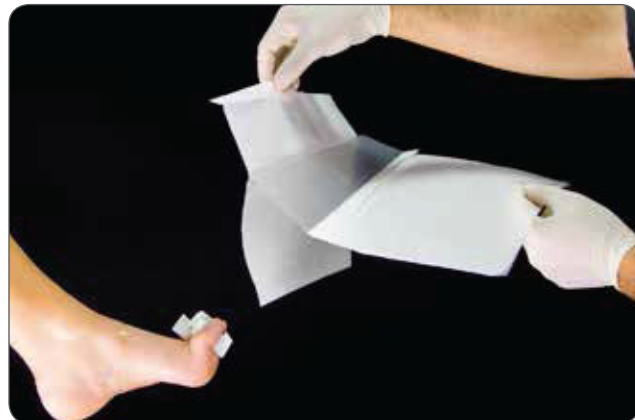

- 7.** After the foot is completely encased in the film, pat down around the edges to ensure a good seal.



- 8.** Carefully remove the second layer of the film backing once on the foot.



ALLEVA MEDICAL CO.

Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan St. Shek Mun, Shatin, N.T., Hong Kong
Tel: +852 3166 7200 Fax: +852 2355 7663 www.allevamedical.com

United States Contact:
Toll-free line at 1-877-312-NPWT
4709 Distribution Ct #2, Orlando, FL 32822
info@allevamedical.com



Application Instructions: (Continued)

- 9.** Once applied, pinch the skin drape over the dressing and cut a hole approximately 2cm wide.



- 10.** Remove the backing layers from the Suction Bell Skirt.



- 11.** Place the Suction Bell opening directly over the hole in the drape with tubing at the desired angle.



ALLEVA MEDICAL CO.

Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan St. Shek Mun, Shatin, N.T., Hong Kong
Tel: +852 3166 7200 Fax: +852 2355 7663 www.allevamedical.com

United States Contact:
Toll-free line at 1-877-312-NPWT
4709 Distribution Ct #2, Orlando, FL 32822
info@allevamedical.com



Application Instructions: (Continued)

- 12.** Attach the connector tubing to the Suction Bell.
Connect the tubing to the extriCARE® NPWT pump.
Power the pump on and set the approximate pressure.



ALLEVA MEDICAL CO.

Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan St. Shek Mun, Shatin, N.T., Hong Kong
Tel: +852 3166 7200 Fax: +852 2355 7663 www.allevamedical.com

United States Contact:
Toll-free line at 1-877-312-NPWT
4709 Distribution Ct #2, Orlando, FL 32822
info@allevamedical.com

